Front panel is shown at approximately 110% of actual size

Kills *C. difficile* spores

**Kills Clostridium
difficile spores**

Kills *C. diff* spores

**Contains
the power of
Clorox® Bleach**

Germicidal Wipes
Toallitas Germicidas

ACTIVE INGREDIENT:
Sodium Hypochlorite 0.55%
OTHER INGREDIENTS: 99.45%
TOTAL: 100.00%
(Available Chlorine 0.52%, 5200 PPM)

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KEEP OUT OF REACH OF CHILDREN
**CAUTION: LIQUID CAUSES
MODERATE EYE IRRITATION.**
SEE BACK LABEL FOR ADDITIONAL PRECAUTIONARY STATEMENTS.

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R0641112

Side panel is shown at approximately 110% of actual size

Germicidal Wipes
Toallitas Germicidas

PATHOGEN/PATÓGENO	CONTACT TIME/TIEMPO DE CONTACTO
Clostridium difficile spores/Esporas de Clostridium difficile	5 minutes/5 minutos
Bacteria/Bacteria	30 seconds†/30 segundos†
‡Viruses/‡Virus	1 minute/1 minuto
Parvoviruses/Parvovirus	5 minutes/5 minutos
Mold, Mildew, Fungi/Hongo, Moho, Fungo	5 minutes/5 minutos

ORGANISMS:

Bacteria:

- †Acinetobacter baumannii ATCC 15308*
- Clostridium difficile* spores ATCC 43598****
- Community Acquired Methicillin resistant Staphylococcus aureus (CA-MRSA)(NARSA NRS123)(Genotype USA400)*
- Community Acquired Methicillin resistant Staphylococcus aureus (CA-MRSA)(NARSA NRS384)(Genotype USA300)*
- †Enterococcus faecalis, vancomycin resistant (VRE) ATCC 51575*
- ESBL (Extended Spectrum Beta Lactamase) producing Escherichia coli ATCC BAA-196*
- †Escherichia coli (E. coli) ATCC 11229*
- Escherichia coli O157:H7 ATCC 35150*
- †Klebsiella pneumoniae ATCC 4352*
- †Legionella pneumophila ATCC 33153*
- †Listeria monocytogenes ATCC 19117*
- †Pseudomonas aeruginosa ATCC 15442*
- †Salmonella enterica ATCC 10708*
- †Shigella dysenteriae ATCC 11835*
- †Staphylococcus aureus ATCC 6538*
- †Staphylococcus aureus, methicillin resistant (MRSA) ATCC 33592*
- †Streptococcus pneumoniae ATCC 6305*
- †Streptococcus pyogenes ATCC 12344*

Viruses:

- †Adenovirus type 2 ATCC VR-846 (Strain Adenoid 6) (common cause of respiratory infections)**
- †Avian Influenza A ATCC VR-2072** , ††
- †Herpes simplex virus type 2 ATCC VR-734 (Strain G) (causative agent of genital herpes)**
- †HIV type 1 (Strain HTLV-IIIB)*

- †Human Coronavirus ATCC VR-740 (Strain 229E) (causative agent of the common cold)**
- †Human Hepatitis A (Strain HM-175)**
- †Human Hepatitis B (as duck HBV)**
- †Human Hepatitis C ATCC VR-1422 (as bovine diarrhea virus)**
- †Influenza A ATCC VR-544 (Strain Hong Kong) (representative of the common flu virus)**
- †Norovirus (as Feline Calicivirus ATCC VR-782)**
- †Poliovirus type 1 ATCC VR-1000 (Strain Brunhilde)**
- †Respiratory syncytial virus ATCC VR-26 (Strain Long) (cause of respiratory infections in infants)**
- †Rhinovirus type 37 ATCC VR-1147 (Strain 151-1) (causative agent of the common cold)**
- †Rotavirus (Strain WA) (causative agent of viral diarrhea)**

Bloodborne Pathogens:

- †HIV type 1 (Strain HTLV-IIIB)*
- †Human Hepatitis B (as duck HBV)**

Parvoviruses:

- Canine parvovirus ATCC VR-2017 (Strain Cornell)****
- Feline panleukopenia virus ATCC VR-648 (Strain Phillips-Roxane) (feline parvovirus)****

Mold, Mildew, Fungi:

- Aspergillus niger ATCC 16404****
- Trichophyton mentagrophytes ATCC 9533****

* 30 second contact time
 ** 1 minute contact time
 **** 5 minute contact time
 †† Kills Avian Influenza A virus on precleaned hard, nonporous surfaces
 † Follow the Special Instructions for Cleaning Prior to Disinfection listed on label

R0641112

Back panel is shown at approximately 110% of actual size

- **One-step disinfectant cleaner designed for general cleaning and disinfecting hard, nonporous environmental surfaces in healthcare facilities**
- **Aids in the reduction of cross-contamination between treated surfaces**
- **Meets OSHA Bloodborne Pathogen Standards**
- **Ready-to-use hospital-grade disinfectant**
- **Ready-to-use disinfection for *C. diff* spores**
- **For daily use against *C. diff* spores**
- **To reduce *C. diff* on surfaces in acute care facilities, long-term care facilities and nursing homes**

DIRECTIONS FOR USE: It is a violation of Federal law to use this product in a manner inconsistent with its labeling. **Not for cleaning or sanitizing skin. Do not use as a diaper wipe or for personal cleansing. For surfaces that may come in contact with food, a potable water rinse is required. Do not use on glassware, utensils or dishes.** Avoid contact with clothes, fabric, wood, natural rubber, and painted and paper surfaces. Do not use on silver or chipped enamel. **APPLY CAREFULLY** — This product contains bleach. **DISPENSER DIRECTIONS:** Remove lid and discard seal. Insert corner of center sheet through flap opening on lid. DO NOT push finger through opening. Replace lid and pull wipe at an angle. The next sheet pops up automatically. When finished, close lid flap to retain moisture.

TO CLEAN, DISINFECT AND DEODORIZE HARD, NONPOROUS SURFACES: Wipe hard, nonporous surface to be disinfected. Use enough wipes for treated surface to remain visibly wet for the contact time listed on label. Let air dry. Gross filth should be removed prior to disinfecting. If streaking is observed, wipe with a clean, damp cloth or paper towel after appropriate contact time has expired. **For Killing Clostridium difficile spores:** Pre-clean hard, nonporous surfaces by removing gross filth, loose dirt, debris, blood/bodily fluids, etc. Wipe hard, nonporous surface to be disinfected. Use enough wipes for treated surface to remain visibly wet for 5 minutes. Rinse. Let air dry. **Special Instructions for Cleaning Prior to Disinfection against Clostridium difficile spores: Personal Protection:** Wear appropriate barrier protection such as gloves, gowns, masks, or eye covering. **Cleaning Procedure:** Fecal matter/waste must be thoroughly cleaned from surfaces/objects before disinfection by application with this product. Cleaning is to include vigorous wiping and/or scrubbing, until all visible soil is removed. Special attention is needed for high-touch surfaces. Surfaces in patient rooms are to be cleaned in an appropriate manner, such as from right to left or left to right, on horizontal surfaces, and top to bottom, on vertical surfaces, to minimize spreading of the spores. Restrooms are to be cleaned last. Do not reuse soiled cloths. **Infectious Materials Disposal:** Materials used in the cleaning process that may contain feces/wastes are to be disposed of immediately in accordance with local regulations for infectious materials disposal. **Special Instructions for Use Against HIV-1, HBV and HCV:** This product kills HIV-1, HBV and HCV on precleaned environmental surfaces/objects previously soiled with blood/body fluids in health care settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus (HIV-1) (associated with AIDS), Human Hepatitis B Virus (HBV) and Human Hepatitis C Virus (HCV). **Special instructions for using this product to clean and decontaminate against HIV-1, HBV and HCV on surfaces/objects soiled with blood/body fluids: Personal Protection:** When handling items soiled with blood or body fluids, use disposable impervious gloves, gowns, masks and eye coverings. **Cleaning Procedure:** Blood and other body fluids must be thoroughly cleaned from surfaces and other objects before applying this product. **Contact Time:** Allow surfaces to remain wet for 1 minute, let air dry. For all other organisms, see directions for contact times. **Disposal of Infectious Materials:** Use disposable impervious gloves, gowns, masks and eye coverings. Blood and other body fluids should be autoclaved and disposed of according to local regulations for infectious waste disposal.

This product is not to be used as a terminal sterilant/high-level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream, or normally sterile areas of the body, or (2) contacts intact mucous membranes, but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semicritical medical devices prior to sterilization or high-level disinfection.

PRECAUTIONARY STATEMENTS: HAZARDS TO HUMANS AND DOMESTIC ANIMALS.

CAUTION:

Liquid causes moderate eye irritation. Do not get in eyes or on clothing. Avoid contact with clothing. Wear protective eyewear. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco or using the toilet. For sensitive skin or prolonged use, wear gloves. **FIRST AID: IF IN EYES:** Hold eye open and rinse slowly and gently with water for 15–20 minutes. Remove contact lenses, if present after the first 5 minutes, then continue rinsing. Call a poison control center or doctor for treatment advice. Have the product container or label with you when calling a poison control center or doctor or going for treatment. **QUESTIONS?** Call 1-888-797-7225. **PHYSICAL AND CHEMICAL HAZARDS:** This product contains bleach. Do not use this product with other chemicals, such as ammonia, toilet bowl cleaners, rust removers or acid, as this releases hazardous gases.

STORAGE AND DISPOSAL:

Store this product in a cool dry area, to avoid deterioration. Dispose of wipe in trash after use. Do not flush. Nonrefillable container. Do not reuse or refill empty container. Recycle empty container or discard in trash. CONTAINS NO PHOSPHORUS.

QUESTIONS OR COMMENTS? Call 1-888-797-7225. For MSDS information, please go to: www.cloroxprofessional.com. A list of this product's ingredients is available at www.CloroxCSR.com.

Mfd. for Clorox Professional Products Company, 1221 Broadway, Oakland, CA 94612. © 2005, 2007 Commercial Solutions is a registered trademark of Clorox Professional Products Company. Clorox is a registered trademark of The Clorox Company. Made in the U.S.A. U.S. Patents 7,008,600 and 7,070,737. EPA Reg. No. 67619-12. EPA Est. No. 8251-WI-5, WI-2, WI-3, WI-4. Actual EPA Est. No. near batch code.

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