

Virucidal*, Bactericidal, Tuberculocidal, Fungicidal, Pseudomonacidal
Clorox Healthcare® EZ-Kill® Quat Alcohol Cleaner Disinfectant Wipes are effective against the following pathogens in 2 minutes at 20°C/68°F, on hard, nonporous surfaces when used as directed:

- Escherichia coli* O157: H7
- Methicillin-resistant *Staphylococcus aureus* (MRSA)
- Mycobacterium tuberculosis*
- Pseudomonas aeruginosa*
- Salmonella enterica*
- Staphylococcus aureus*
- Vancomycin-resistant *Enterococcus faecalis* (VRE)
- *Herpes Simplex Virus 2 (genital herpes virus)
- *HIV-1 (AIDS virus)† (at an exposure time of 1 min.)
- *Human Hepatitis B Virus
- *Human Hepatitis C Virus
- *Influenza A virus (A2 Japan)

†See back panel for complete HIV-1 (AIDS virus) information.

Clorox Healthcare® EZ-Kill® Quat Alcohol Cleaner Disinfectant Wipes are effective against the following pathogens in 5 minutes at 20°C/68°F, on hard, nonporous surfaces when used as directed:

- Trichophyton mentagrophytes*
 (Athlete's Foot Fungus)



EZ-KILL®

Quat Alcohol Cleaner Disinfectant Wipes

ACTIVE INGREDIENTS:

| | |
|--|---------------|
| N-Alkyl (60% C14, 30% C16, 5% C12, 5% C18) | |
| Dimethyl Benzyl Ammonium Chloride | 0.12% |
| N-Alkyl (68% C12, 32% C14) | 0.12% |
| Dimethyl Ethyl Benzyl Ammonium Chloride | |
| Isopropyl Alcohol | 41.58% |
| OTHER INGREDIENTS*: | <u>58.18%</u> |
| TOTAL: | 100.00% |

† Does not include weight of dry wipe.

KEEP OUT OF REACH OF CHILDREN
WARNING

DO NOT USE AS A BABY WIPE

See back panel for additional precautionary statements.

65 Pre-Saturated Wipes (10 in. x 10 in.)
NET WEIGHT: 1 lb. 10 oz.

For Use on Environmental Surfaces.

Areas of Use:

- | | | |
|----------------------------|----------------------|-----------------------|
| ambulances | exam rooms | patient rooms |
| anesthesia | hospital | recovery |
| bathrooms | intensive care units | respiratory therapy |
| cat. lab | kitchens | surgery |
| clinics | laboratories | veterinary facilities |
| dental offices | medical offices | x-ray |
| emergency medical services | newborn nursery | |
| emergency rooms | nursing facilities | |
| | orthopedics | |

PREPARATION OF DISPENSER: Remove cover from canister and discard covering seal. From center of wipes roll, pull up a wipe corner, twist into a point and thread through the tear throat in the underside of the lid cover. Pull wipe through about one (1) inch. Replace the lid cover on container, assuring a secure fit. Pull out first wipe and snap at a 90° angle. Remaining wipes feed automatically for next use. Always reseal hinged cap after each use to ensure moisture retention.

Reorder # 7105
 NI-36301



DIRECTIONS FOR USE: It is a violation of U.S. Federal law to use this product in a manner inconsistent with its labeling.

This product is not to be used as a terminal sterilant/high-level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes, but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization or high-level disinfection. This product is not for use on the inhalation therapy and/or pulmonary diagnostic equipment.

DISINFECTION: Thoroughly wet pre-cleaned, hard, nonporous surface with a wipe, keep wet for 2 minutes (5 minutes if fungus is suspected), and allow to air dry. Use as many wipes as needed for the treated surface to remain wet for the entire contact time. For use as a general disinfectant wipe on hard, nonporous, inanimate surfaces. You may use a wipe to pre-clean the surface to be disinfected, but thorough re-wiping with a fresh wipe is required for disinfection. Use on food-contact surfaces must be followed by a potable water rinse.

For use on hard, nonporous surfaces only, such as:

- | | | |
|----------------------------------|---|--------------|
| ambulance equipment and surfaces | external surface of respiratory and/or anesthetic equipment | switches |
| basins | Formica® frames | telephones |
| baskets | glass | toilets |
| bedside cabinets | laundry hampers | trash carts |
| carts | light fixtures | tubs |
| chairs | mirrors | urinals |
| chart racks | office furniture | walkers |
| combs | operating room hard surfaces | waste cans |
| countertops | plastic | wastebaskets |
| counters | sinks | wheel chairs |
| doorknobs | stainless steel | |
| equipment | | |
| exam tables | | |

DEODORIZING: Wipe hard, nonporous surface thoroughly to eliminate odors from bathrooms, fecal matter, urine, burns, smoke, decay, stale air, tobacco, and sickrooms.

CLEANING: To clean surfaces, wipe thoroughly with a fresh wipe, which may be used to deodorize at the same time as cleaning. When used to clean a hard, nonporous surface prior to disinfection, a second wipe must be used for disinfection.

THE ACTIVITY OF Clorox Healthcare® EZ-Kill® QUAT ALCOHOL CLEANER DISINFECTANT WIPES AGAINST THE HIV-1 (AIDS VIRUS):

Clorox Healthcare® EZ-Kill® Quat Alcohol Cleaner Disinfectant Wipes completely inactivate the HIV-1 (AIDS virus) on hard, nonporous surfaces when used at a contact time of 1 minute at room temperature (20°–25°C).

Clorox Healthcare® EZ-Kill® Quat Alcohol Cleaner Disinfectant Wipes kill HIV-1 on pre-cleaned environmental surfaces/objects soiled with blood/body fluids, in healthcare settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of human immunodeficiency virus Type-1 (HIV-1) (associated with AIDS).

SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 ON SURFACES/OBJECTS SOILED WITH BLOOD/BODY FLUIDS:

Personal Protection: Specific barrier protection items to be used when handling items soiled with blood or body fluids are disposable latex gloves, gowns, masks and eye coverings. **Cleaning Procedure:** Blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of Clorox Healthcare® EZ-Kill® Quat Alcohol Cleaner Disinfectant Wipes. **Contact Time:** HIV-1 virus is inactivated in 1 minute.

However, other pathogens require a 5-minute exposure for complete kill. See disinfection directions.

Disposal of Infectious Materials: Materials that may contain blood or other body fluids should be autoclaved and disposed of according to federal, state and local regulations for infectious waste disposal.

PRECAUTIONARY STATEMENTS:
 Hazards to Humans and Domestic Animals

WARNING: Causes substantial, but temporary eye injury. Do not get in eyes or on clothing. Wear specific, appropriate protective eyewear such as goggles, face shield or safety glasses. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum or using tobacco. Wear disposable latex gloves, gowns, masks and eye coverings when handling HIV-1 infected blood or body fluids. Avoid contamination of food. Remove and wash contaminated clothing before reuse.

FIRST AID:

If in eyes: Hold eye open and rinse slowly and gently with water for at least 15–20 minutes. Remove contact lenses, if present, after first 5 minutes, then continue rinsing eye. Call a poison control center or doctor for treatment advice. **If on skin or clothing:** Take off contaminated clothing. Rinse skin immediately with plenty of water for 15–20 minutes. Call a poison control center or doctor for treatment advice. Wash contaminated clothing before reuse. **WASH THOROUGHLY AFTER HANDLING WIPES.**

STORAGE AND DISPOSAL:

Store in cool area. Keep away from heat or open flame. Do not reuse wipes. Dispose of used wipe in trash. **Container Disposal:** Non-refillable container. Do not reuse or refill this container. Offer for recycling if available.

EPA Reg. No. 59894-10-67619 EPA Est. No. 73027-WI-1
 For technical information, call 1-800-638-2625. Contact 1-800-255-3924 for emergency medical treatment information. For SDS, go to cloroxhealthcare.com
 Manufactured for Clorox Professional Products Company,
 1221 Broadway, Oakland, CA 94612.
 © 2016 The Clorox Healthcare logo and EZ-Kill are registered trademarks of Clorox Professional Products Company. All rights reserved.